SINCE 1992

Non-contact Infrared Thermometer BODY COMFORT BC 07
User's manual / EN (page 2-10)

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**Non-contact Infrared Fever Thermometer BODY
COMFORT BC 07
User's manual**

Please read this manual carefully and follow all instructions inside before first use !

Contents

1.General Description	3
2.Important Informations	4
3.Features	4
4.Intended Use	5
5.Description of Thermometer	5
6.Description of Symbols on Display	6
7.Explanation of Signs and Symbols	6
8.Technical Specifications	6
9.Advices Before 1 st Use	7
10.Measuring Operation	7
11.Maintenance and Cleaning	9
12.Troubleshooting	9
13.This Thermometer is Designed and Produced according to Following Standards	9
14.Guarantee	10

Important note: English language text of this instruction manual is valid only in english speaking countries.

1. General Description

Congratulations on buying your NON-CONTACT medical fever thermometer BODY COMFORT BC 07. This thermometer is especially designed to take the human body temperature or temperature of the baby milk, baby food and others.

Depending on various skin types and thickness, there may be temperature differences.

2. Important Information

- This non-contact fever thermometer should only be used for the purposes described in this instruction manual
- This non-contact fever thermometer is adjusted and calibrated in the production, so there is no need of any additional adjustment, regulation or similar
- According to many clinical research studies, without complying with specific regulations, there is no chance to compare the measurement results made on different types of thermometers and on different parts of human body
- This thermometer can be used in an ambient temperature range between + 10 to 40 degree Celsius only
- Do not expose this thermometer to the electric shocks and to extreme temperature conditions of higher than 50 degree Celsius or lower than 0 degree Celsius
- Do not store or use this thermometer on relative humidity higher than 85 %
- With this fever thermometer you can make the body temperature measurement on following body parts: behind the ear lobe, inside the ear, on the temple, orally, rectally, axillary, forehead
- After inserting a new batteries it is necessary to wait approximately 10 minutes prior to first measurement
- Time distance between each following measurement must be at least 30 seconds!
- Do not use this thermometer near large electromagnetic fields such as mobile phones and similar
- Keep this thermometer away from water or any liquid, away from heat or direct sunlight
- Do not drop or knock this thermometer and do not use it damaged
- Measurement accuracy can be affected by many factors (for instance when the forehead is covered by hair, perspiration, cap or scarf)
- Keep the measuring distance 1 – 10 centimeters only!
- This thermometer should be left in the room 15 – 20 minutes before measurement
 - Protective glass in front of the thermometer must be kept clean at all times !
 -

NEVER focus this thermometer towards the eyes!

- Select “BODY” mode to measure the body temperature
- Select “SURFACE” mode to measure the surface temperature
- In case of any problems or uncertainties do not open this thermometer, please do not hesitate to contact the authorized service center, of which address could be found in this instruction manual.

3. Features

- Precise non-contact measurement of human body temperature, but also surface (certain surfaces) temperatures
- Selectable BODY or SURFACE temperature
- Automatic memorization of last 32 measurement results
- Set Alarm adjustment
- Automatic switch off
- Selectable Celsius or Fahrenheit temperature range
- Display resolution 0,1 degree
 - Backlight LCD display

4. Intended Use

This non-contact infrared medical fever thermometer is designed for body temperature measurement for infants and adults in "BODY" mode, without contact to human body. Measurement environment range is 1 – 60 degree Celsius.

This thermometer can also be used to measure the temperature of baby-bottle, bath, or room temperature by selecting the "SURFACE" measurement mode.

Standard temperatures according to WHO (World Health Organization)

<i>Measurement method</i>	<i>Normal Temp. in degree Celsius / in Fahrenheit</i>	
Rectal	36,6 – 38	97,8 – 100,4
Oral	35,5 – 37,5	95,9 - 99,5
Axillary	34,7 – 37,3	94,4 - 99,1
Ear	35,8 – 38	96,4 - 100,4


Temperature of the human body varies throughout the day. It can also be influenced by many external factors like for example: age, sex, type and thickness of the skin and others.

Normal temperature according to age:

<i>Age in years</i>	<i>Temperature in degree Celsius / Fahrenheit</i>	
0 – 2	36,4 – 38	97,5 - 100,4
3 – 10	36,1 – 37,8	97,0 - 100,0
11 – 65	35,9 – 37,6	96,6 - 99,7
Over 65 years	35,8 – 37,5	96,4 - 99,5

5. Description of Thermometer BODY COMFORT BC 07

1. Infrared Sensor
2. LCD Display
3. "BODY" or "SURFACE" selection
4. Down button
5. Up button
6. Mode button
7. Measurement trigger
8. Battery cover

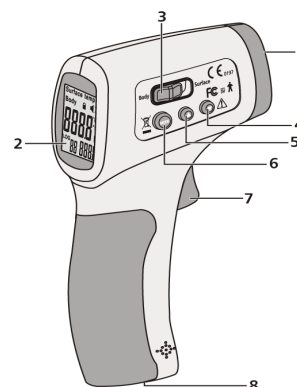6. Description of Symbols on Display

1. Surface mode symbol
2. Body mode symbol
3. Digital readout
4. Low battery symbol
5. The order number – memory
6. Save data readout
7. Temperature scale Celsius or Fahrenheit
8. Buzzer symbol

7. Explanation of Signs and Symbols

CE 0197

This non-contact fever thermometer is in accordance with Medical Device Directive 93/42/EEC

	The device is in accordance with Medical Device Directive 93/42/EEC
	The device is in accordance with FCC Part 15 Subpart B:2007/Radio Frequency Devices IC Regulation ICEC-003: 2004 Interference-causing Equipment Standard-Digital Apparatus
	3V DC power supply
	Type B equipment
	In order to protect the environment, please recycle the battery according to the local regulations
Body Surface	Indication of Mode
	Attention, consult accompanying documents

8. Technical specifications

Normal Conditions of Use

Display Resolution	0.1°C (0.1°F)
Operating Temperature	10 to 40°C (50 to 104°F)
Storage Temperature	0 to 50°C (32 to 122°F)
Humidity Rate	≤ 85%
Power	DC 3V (2x “AAA” batteries)
Size	128 x 74 x 36 mm / 5 x 2.9 x 1.4 in (LxWxH)
Weight	Gross 125.4g / Net 104.5g

Measuring Range

In Body Mode	32.0 to 42.5°C (90 to 108°F)
In Surface Temp Mod	0 to 60°C (32 to 140°F)
Accuracy	± 0.3°C (0.54°F)
Measuring Distance	1 – 10 cm (0.4 – 3.94 in)
Automatic Stop	7 sec


Non-contact BODY COMFORT Infrared Thermometer Measurement Precision

32 – 35.9°C / 93.2 – 96.6°F	± 0.3°C (0.5°F)	According to ASTM
36 – 39°C / 96.8 – 102.2°F	± 0.2°C (0.4°F)	Standard E1965-1998
39 – 42.5°C / 102.2 – 108.5°F	± 0.3°C (0.5°F)	(2003)

9. Advices Before 1st Use

It is very important to follow instructions below, before starting a first measurement:

- After inserting the batteries into the battery compartment with the right polarization + and – it is necessary to wait at least 10 minutes before 1st measurement,
- Check the forehead or other places of the measured person and make sure the measured body part of is clean and without any perspiration, hair and others, what could cause wrong measurement results,
- Check if the right measurement mode is selected, “BODY” mode for body temperature or “SURFACE” mode for baby milk or similar
- Focus the thermometer on the selected body part central part for measurement, -Remember the right measurement distance 1 – 10 centimeters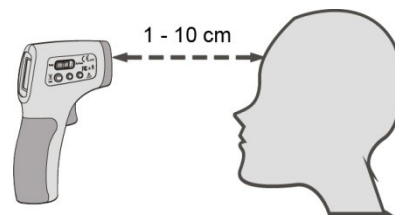

10. Measuring Operation

This non-contact medical infrared fever thermometer BODY COMFORT BC 07 is designed and produced especially for measuring body temperature in ear, temple, behind the ear lobe, forehead only, all without any contact to the body, in environmental temperature range described in this instruction manual. It can be used also for measurement of baby-bottles and similar.

How to use properly this thermometer? Press the measurement trigger while focusing the thermometer on the selected body parts, and there will be a beeping signal produced by the thermometer after around 2 seconds, what means the measurement is completed. Watch the back side of the thermometer, where the measurement result will be displayed for few seconds. Measurement can be repeated after 30 seconds, by doing the same as before - push the measurement trigger again and follow the same steps as described above.

Additional functions of this thermometer:

-a- Choosing temperature range C/F – Function F 1

Press “MODE” button for 2 seconds and the screen displays F1. Select „DOWN” for degree Celsius , or select “UP” for degree Fahrenheit.

-b- Alarm selection F2 Menu

Press “MODE” button for 2 seconds and on the screen will be displayed F1.

Press “MODE” twice to get F2. Select “UP” to increase the threshold by 0,1 degree Celsius, or select “DOWN” to reduce it by 0,1 degree Celsius.

NOTE: The pre-adjusted alarm threshold default value is 38 degree Celsius.

To finish any adjustment press “MODE” button until the display will be switched off.

The body temperature is usually different from room or environment temperature.

Data Memory

Press “UP” or “DOWN” button to display the last temperature measurement.


When the thermometer is off, press “UP” and “DOWN” button together for 2 seconds to display the last temperature measurement.

At this mode if change the order number to 0 and press “MODE” button, then delete all memory data.

If you change the order number to “0” and press “MODE” button at this mode, then all memory data will be deleted.

Changing the Batteries

It is necessary to use 2 pcs of non-rechargeable 1,5 Volt AAA batteries. When the LCD screen displays the sign of low battery, open the lid and change the batteries taking a great care with the correct positioning of + and – of battery polarity ! A mistake with this could cause damage of the thermometer and compromise the guarantee of this thermometer. Never use rechargeable batteries! Use only batteries for single use! If the thermometer is not in use for longer time, remove the batteries from the thermometer to extend the usage time and avoid any damage resulting from leaking batteries.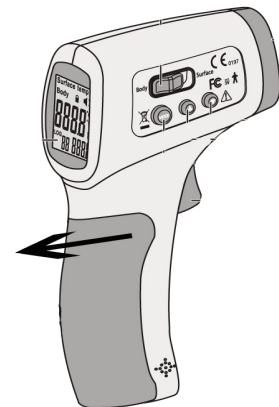

Durability of this Thermometer

This non-contact infrared Thermometer was designed for intense and long-term use. Its durability is adjusted to approximately 40.000 measurements.

11. Maintenance and Cleaning

The protective glass over the lens is the most important and fragile part of the thermometer, please take great care of it.

The infrared sensor is the most precise part, so it must be protected carefully.

Clean the thermometer with a cotton bud lightly moistened with 70 % alcohol.

Do not clean the thermometer with corrosive detergent.

Keep the device away from water or other liquids.

Store the device in a dry and clean environment keep it away from dust and direct sunlight.

12. Troubleshooting

While using this thermometer could happen to display on the screen following symbols “HI” or “Lo”.



-a- When is on the screen displayed “HI”, it means, the measurement result is higher than 42,5 degree (in “BODY” mode) or 60 degree Celsius (in “SURFACE” mode)

-b- When is on the screen displayed “Lo”, it means, the measurement result is lower than 32 degree

(in “BODY” mode) or 0 degree Celsius (in “SURFACE” mode) .

If the problem persists, please contact the authorized service.

Reasons for “LO” or “HI” message display	Advice
Temperature reading hampered by hair, perspiration...	Make sure that there is no obstruction prior to taking a temperature.
Temperature hampered by an air flux.	Make sure there is no air flux as this could interfere with the infrared system.
The measuring distance is too far.	Please respect the measuring distance (1 - 10 cm)
From high/low temperature condition to room temperature	Waiting for 10 minutes before taking the body temperature

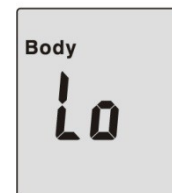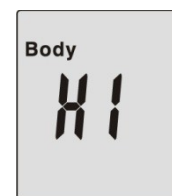13. This Thermometer is designed, developed and manufactured according to Following Standards

Non-contact infrared fever thermometer BODY COMFORT BC 07 is designed, developed and pmanufactured according to following standards:

- MDD 93/42/EEC + CE 0197
- EN 980 : Graphical symbols for use in the labeling of medical devices
- EN 1041 : Information supplied by the manufacturer with the medical devices
- EN 60601 : Medical electrical equipment Part 1: General requirements for safety / IEC 60601:1:1998 /
- EN 60601-1-2: Medical electrical equipment Part 1-2:General standards and requirements for electromagnetic compatibility
- EN 60601-1-2:2007 for EMC
- ASTM Standard E 1965-1998 /2003/

The manufacturer reserves the right to alter the specifications of the product without prior notice. The manufacturer allows himself the right to modify the technical specifications of this thermometer without any preliminary opinion or prior notification.

14. Guarantee

Manufacturer of non-contact infrared fever thermometer BODY COMFORT BC 07 is fully responsible for all product damages caused by production only, in home individual use within 2 years after the date of the purchase. In any case of any professional or medical use, the guarantee period is 6 months after the date of the purchase or invoice issuance. During this date period manufacturer is responsible to solve the product damages caused only by the production for free. Any transportation fee is to account of the customer.

This guarantee does not include any damages caused by user, unauthorized staff or similar. Manufacturer is not responsible for damages caused through non-proper use of this thermometer for different purpose as described in this instruction manual. In any case of any non-authorized hit or intervention into this thermometer, fall out any right or requirement for any guarantee. To apply successfully for the eventual guarantee service is necessary to present the original bill or invoice of the authorized seller.

In case of any claims, queries or uncertainties, please dont hesitate to contact the following authorized service: info@bodycomfort.biz

Date of purchase:

Stamp and signature of authorized seller:

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